



Device	MR5 ACTIVet PRO				
Market	VET				
	Wavelength	Type	# of Diodes	(P)ower/each	Total (P)
Infrared	905	SPL	1	50 W	.02 to 20 mW
Infrared	860	Broadband	3	80 mW	250 mW
Infrared	N/A				
Red	660	LED	3	65 mW	200 mW
Blue	465	LED	3	80 mW	250 mW
Magnet	N/A	N/A	1	35 mT	35 mT
Dose per minute	27J/min				
Aperature	4 cm2				
Power Source	Battery				
Density	112.5 mw/cm2 to		117.5 mw/cm2		
Times	.5, 1, 2, 3, 4, 5, 10, 15, 20				
Probes	Yes, included				
Configuration	1:3:3:3				
Joules Counter	Yes				
Core Frequencies	5, 50, 1000, 1000-3000, 3000				
Additional Frequencies	250, 500, 3000, 5000, 5-50, 5-250, 5-500, 5-1000, 500-1000, 500-5000				
Power Settings	ACTIVET mode (25% output of all light sources)				
Favorites Mode	Yes, 10				
Manual Mode	Yes, All Light sources, ON/OFF, Red and Blue CW or 2 Hz				
Continuous "Mode"	RED and BLUE Only				
Red + Blue Simultaneously	In Manual Mode				
Electrical Stimulation	N/A				
Other Modes	Blue, Equine, Companion (Small), Companion (Large), Saved Programs				



Start Up:

1. Plug in adapter to charge inbuilt battery
2. Power ON: Hold the **START/STOP**
3. Power OFF: Hold the **START/STOP**
4. Battery indicator shows the current level of charge remaining
5. Self-Test: An optional test that checks the operation of each of the 4 light sources



Navigation:

1. Select Mode: Press **SELECT** to enter the Mode
2. Press ◀ or ▶ to cycle through all available Modes
3. To return to a prior screen, option, or the Main Menu, press the **RETURN** key on any screen
4. Press the **START/STOP** to initiate/terminate treatment

ACTIVET PRO	ACTIVET	BLUE MODE	EQUINE	COMPANION	SAVED PROGRAMS
Most frequently used mode Power is set to 100% Able to manually select all parameters (Manual Settings) is available in this mode ONLY	Original 6 settings from ACTIVet No blue light option Power is set to 25% No modification can be made to the settings No Energy Delivered Counter	Anti-microbial blue light therapy Delivers 15 J per minute Power is set to 100% and Continuous Wave	10 Presets based on the Priority Principle Symptom based Simplest and easiest way to get results	10 Presets Protocols on the Priority Principle modified by the size of the animal Large >20 lbs., Small <20 lbs. Symptom based Simplest and easiest way to get results	Up to 10 Programs can be saved in this mode Programs are built in the ACTIVET PRO Mode ONLY
Joules		Joules	Joules	Joules	

Start a Treatment:

1. Enter *desired Mode*
2. Select *Frequency*
3. Press ◀ or ▶ to cycle through all available Frequencies
4. Press **Select** to Enter
5. *Select Time*
6. Press ◀ or ▶ to cycle through all available Times
7. Press **Select** to Enter
8. To start Treatment, press **START/STOP**

Start Priority Principle Treatment:

1. Select *Companion – Large, Companion – Small, or Equine*
2. Select the **Priority**
3. Press **START**

To Display Joules Counter:

1. During active treatment, press the **RETURN** key to toggle between Joules delivered and Time remaining.
2. Works in ACTIVet PRO and Priority Principle Modes ONLY

Photoprobes:

1. **General Utility Probe:** intraoral, intranasal, focusing on articular surfaces, muscle spasm, etc.
2. **Corporal Probe:** Trigger points, corporal acupuncture points
3. **Wound:** Prevents debris from entering the aperture, assist with scanning
4. **Auricular:** pinpoint acupuncture points
5. **Hair:** For thick coats to facilitate

Manual Mode: Sub Menu

1. Follow steps above to *Start a Treatment*
2. Before starting treatment, press **SELECT** to Enter Sub Menu Options
3. Press **SELECT** to cycle through all available Options
4. Press ◀ or ▶ to toggle the Options
5. There are 5 Option:
 - Blue: On or Off
 - Red: On or Off
 - IRL: On or Off
 - IRED: On or OFF
 - LED Mode: PW or CW
6. To start Treatment, press **START/STOP**